

# EzPAP® Positive Airway Pressure Therapy System Set-Up Guidelines

## 1 Set Flow Rate

Begin at 5 LPM [fig 1]. Increase until desired PAP is reached. (Do not exceed 15 LPM.)

## 2 Verify Pressure

Should not exceed 20cm H<sub>2</sub>O [fig 2].

## 3 Breathe

Patient should be instructed to inhale and exhale slowly [fig 3].

NOTE: Refer to individual product instructions for use for warnings, cautions, contraindications, and additional information

EzPAP® with nebulizer

